DYNAMIC ANKLE SUPPORT

GMN: 872076918P03NB

Inhoud verpakking

TriGlide
Maat: SKU: UDI-DI:
XS/L 3GL25 8720769180486
XL 3GL38 8720769180479

Bandbeugel montageset
Maat: SKU: UDI-DI:
XS/L BB25 8720769180592
XL BB38 8720769180615

Dynamic Ankle Supports (L+R)

Ø:	Maat:	SKU:	UDI-DI:
140-170mm	XS	P03XS	8720769180172
170-200mm	S	P03S	8720769180561
200-230mm	M	P03M	8720769180578
230-280mm	L	P03L	8720769180585
280-330mm	XL	P03XL	8720769180608

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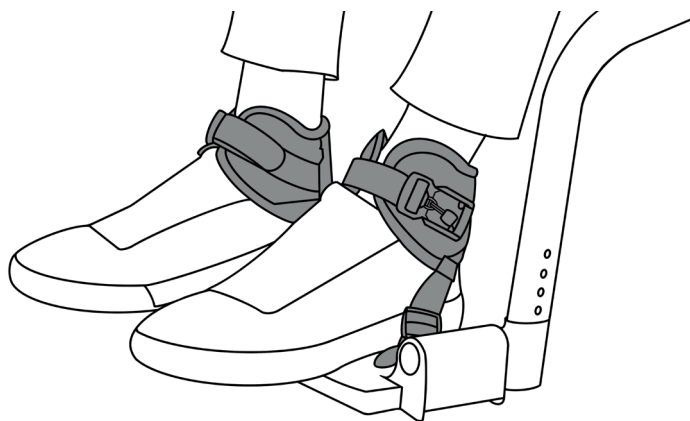
NO SAFETY DEVICE FOR TRANSPORT, OR RESTRAINT
Solely intended for foot / ankle support of persons in a wheelchair. Read the instructions before installation and use.

The Dynamic Ankle Supports should be measured and applied over the daily footwear and, if present, ankle-foot orthoses (AFOs). **Note: shoes with heavy insoles or other orthopedic devices may prevent proper fitting.** Suitable for any type of foot support, provided the positioning needs of the user are always taken into account.

PREVENT INJURIES AND SKIN DAMAGE. This product can alter the sitting posture. Use only on persons wearing a properly fitting pelvic positioning belt. Regular pressure relief and monitoring of pressure points are necessary, especially at contact areas such as the sacrum, legs, and ischial tuberosities. If redness or skin irritation occurs, discontinue use and consult a doctor or wheelchair specialist.

PERIODIC INSPECTION: For safety, this product must be regularly checked for functionality and wear. Discontinue use if there are defects or wear on buckles, mounting points, straps, cushions, or stitching, and contact your supplier for repair or replacement. Never allow unqualified persons to modify or repair this product.

DYNAMIC ANKLE SUPPORT

GMN: 872076918P03NB

L

Package contents

4x

TriGlide

Size: SKU: UDI-DI:

XS/L 3GL25 8720769180486

XL 3GL38 8720769180479

4x

Band bracket mounting set

Size: SKU: UDI-DI:

XS/L BB25 8720769180592

XL BB38 8720769180615

R

Dynamic Ankle Supports (L+R)

Ø:	Size:	SKU:	UDI-DI:
140-170mm	XS	P03XS	8720769180172
170-200mm	S	P03S	8720769180561
200-230mm	M	P03M	8720769180578
230-280mm	L	P03L	8720769180585
280-330mm	XL	P03XL	8720769180608

Assembly

If the wheelchair footrests are equipped with slots, the straps can be threaded through.

Thread the strap through the slot or strap bracket, and through the TriGlide in sequence 1, 2, 3, 4. Adjust the length of the strap so that the sole of the foot is in contact with the wheelchair footrest.

Ø 5mm

+

Use the supplied band bracket mounting set for footrests without slots. Mark the holes before drilling and mount the band brackets using the M5 screws and nuts.

- 1
- 2
- 3
- 4

General use

The Dynamic Ankle Supports should be worn snugly around the ankle (above the foot). The Fidlock buckles can be positioned on the inside or outside of the foot for easy accessibility.

Dynamic Support

The degree of tension on the elastic foot straps determines the range of motion.

Maintenance

Hand or machine wash at max. 30°C using a mild detergent. Do not iron, tumble dry, or centrifuge.

Disposal

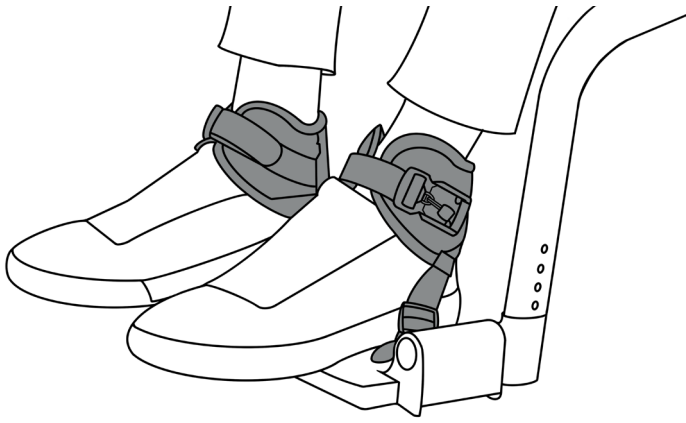
Made from materials that can be safely disposed of at the end of use, without requiring special precautions.

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© 2025 - All rights reserved