**MODULAIRE POSITIONERINGSGORDEL**

GMN: 872076918P01N7

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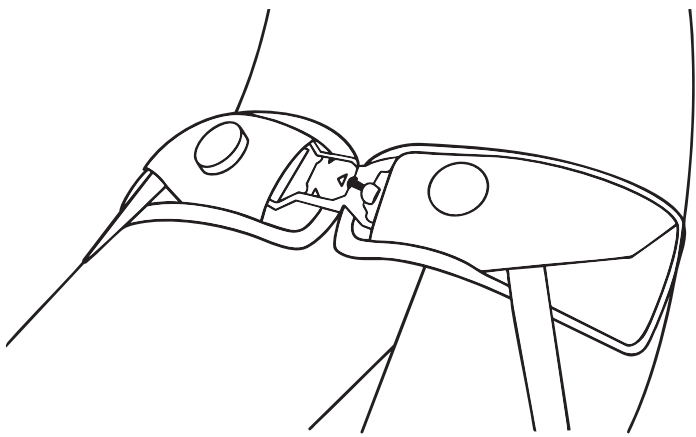
**MODULAR PELVIC POSITIONING BELT**

GMN: 872076918P01N7

**⚠ This product is solely intended for positioning or supporting a person in a wheelchair or work chair. Not suitable as a safety belt in vehicles or for restricting movement. Improper use is unauthorized and unsafe.**

This product must be installed by a qualified rehabilitation technician. Use the optional Posigo® mounting kit for installation on wheelchairs or seating systems.

Always tighten the positioning belt securely over the pelvis or thigh. Have your rehabilitation specialist demonstrate correct use and adjustment.

**A loose belt can lead to slippage and choking hazards. A detached belt can cause falls or slippage. If the occupant cannot release the belt independently, this may be dangerous, especially in emergencies.**

In cases of limited physical or cognitive abilities, an attendant must always be present to operate the belt safely. If a serious incident occurs due to the use of this product, it must be reported to the manufacturer (The Hague Care Development BV) and the local competent authority.

**PREVENT INJURIES AND SKIN DAMAGE. This product may alter the seating posture. Regular pressure relief and inspection of pressure points are essential, especially in contact areas such as the sacrum, legs, and ischial tuberosities. If redness or skin irritation occurs, discontinue use and consult a doctor or wheelchair specialist.**

**PERIODIC INSPECTION:** To ensure occupant safety, this product must be regularly checked for functionality and wear. Do not use the product if it is not functioning properly or if there is visible wear on the buckles, mounting points, straps, padding, or stitching. In such cases, contact your Posigo® supplier for professional repair or replacement.

The product must never be adjusted or repaired by unqualified personnel. The health and safety of the occupant depend on this!

**Warranty:**

This product comes with a limited lifetime warranty against material and manufacturing defects under normal use. Contact your supplier or The Hague Care Development BV for warranty claims.

**Maintenance:**

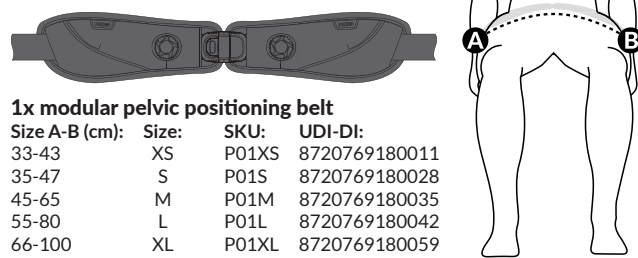
Hand wash or machine wash at a maximum of 30°C using delicate detergent if necessary. Do not iron, spin dry, or tumble dry.

**Waste/Disposal**

Made from materials that can be safely disposed of at the end of their useful life without special precautions.

**Packaging Contents:**

**1x modular pelvic positioning belt**

Size A-B (cm):	Size:	SKU:	UDI-DI:
33-43	XS	P01XS	8720769180011
35-47	S	P01S	8720769180028
45-65	M	P01M	8720769180035
55-80	L	P01L	8720769180042
66-100	XL	P01XL	8720769180059

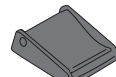
**2x 4-point straps**  
SKU: 4P20  
UDI-DI: 8720769180110

**2x TriGlide**  
20/38mm

Size: SKU: UDI-DI:  
XS 3GL20 8720769180493  
S-XL 3GL38 8720769180479


**2x cam buckle (belt)**  
25/40mm

Size: SKU: UDI-DI:  
XS KG25 8720769180134  
S-XL KG40 8720769180127

**2x cam buckle (4-point strap)**  
25/40mm

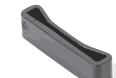SKU: UDI-DI:  
KG20 8720769180547

**2x belt guide (belt)**  
25/38mm

Size: SKU: UDI-DI:  
XS BG25 8720769180523  
S-XL BG38 8720769180509


**2x belt guide (4-point strap)**  
20mm

SKU: UDI-DI:  
BG20 8720769180516


**Installation on Wheelchair/Seating System:**

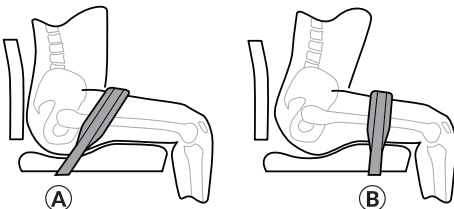Determine the optimal position and angle (A, B) based on clinical goals and the occupant's positioning needs.

For the management of a tendency towards pelvic posterior tilt, the recommended belt positioning is:

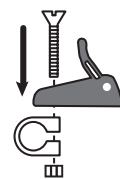
Attach cam buckles at the desired position on the wheelchair, following the wheelchair manufacturer's instructions.

Mounting of cam buckle on round tube clamp (optionally available).

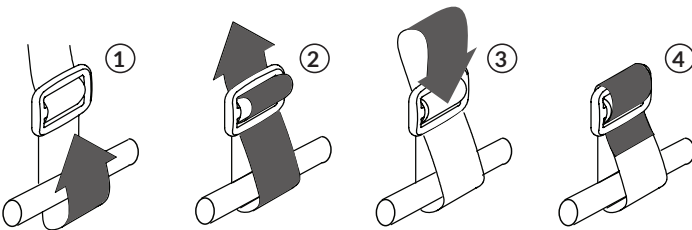
If the wheelchair is unsuitable for tube clamps or cam buckles, use the provided TriGlides. Loop the strap through the TriGlide in order 1,2,3,4 and around the frame, tightening securely.


Regularly check the straps and TriGlides for tightness.

**4-Point Configuration (Optional):**

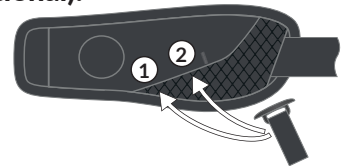Hook the T-connection of the 4-point strap into the loop in the cavity of the padding at the desired position (1, 2) and ensure it is securely fastened. Note: The XS belt has only 1 cavity.

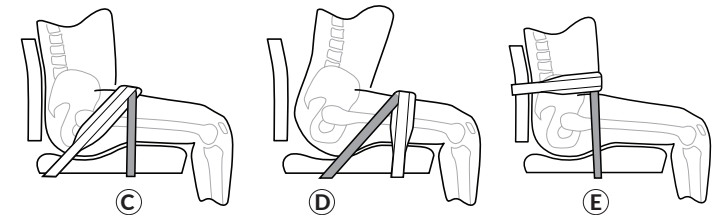
Attach the ends of the 4-point straps to the wheelchair with the provided 20mm cam buckles. Position the cam buckle (C, D, E) in consultation with the therapist.

For the management of a tendency towards pelvic anterior tilt, the 4-point configuration is recommended with belt positioning E


**First Use/Adjustment:**

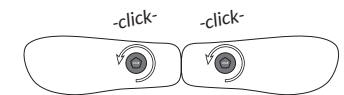Rotate the adjustment knobs fully counterclockwise until they click, extending the straps to their maximum length.


Center the belt on the occupant (refer to the size markings on the straps).

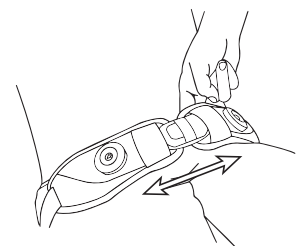Pass the strap ends through the cam buckles attached to the wheelchair frame until the belt fits snugly on the occupant. Close the cam buckles.

Ensure the tension is appropriate by inserting a finger between the padding and the occupant.

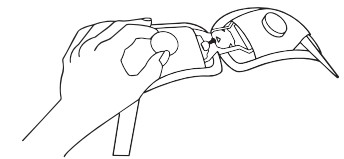During use, the fit can be adjusted using the adjustment knobs. Turning clockwise increases tension, while turning counterclockwise provides slack. Before opening the belt, turn the knobs counterclockwise to release tension.


**Replacing Buckles:**

Optionally available buckles:

Push-buckle	Pin-buckle	FidLock buckle	Side release buckle
XS: CBUT25	XS: CPIN25	XS: CFID25	XS: CSID25
S-XL: CBUT40	S-XL: CPIN40	S-XL: CFID40	S-XL: CSID40

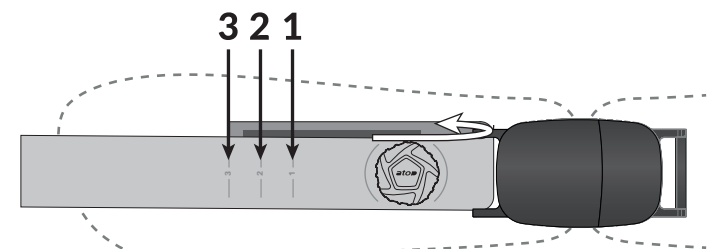Slide the padding over the adjustment knob so that the nylon strap with the hook & loop becomes visible. Pull the hook & loop apart and remove the buckle. Attach the replacement buckle in reverse order. **Note: The nylon strap is marked with a scale 1, 2, 3, which determines the length of the loop. Position 3 corresponds to the push- and pin buckle, position 2 to the FidLock buckle, and position 1 to the side-release buckle. Only replace the buckles with original ones provided by Posigo®.**

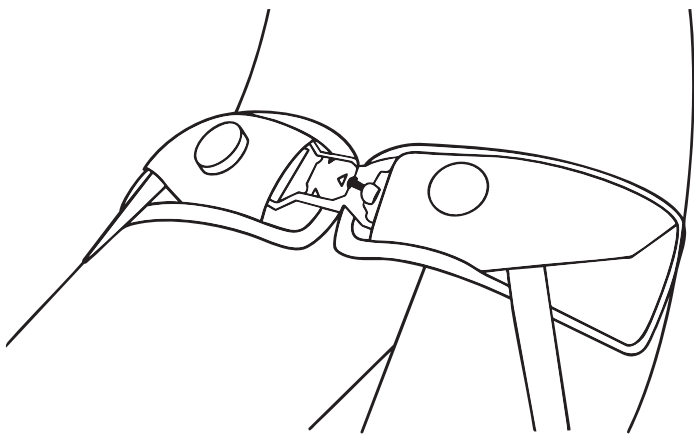
The back of the nylon strap is equipped with hook & loop.

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© 2025 - All rights reserved